

Andarta Body Fluid Spill Kit


Each spill kit includes:

- 1 x Disposable Gloves
- 1 x Disposable Apron
- 1 x Disposable Odour Mask
- 1 x Absorbent Pad
- 1 x Scoop & Spatula
- 6 x Wipes
- 1 x Waste Bag & Ties

Once the spill has been appropriately dealt with, place the used scoop, scraper and apron into the yellow waste bag and tie securely.

Dispose of the contaminated materials in an approved way.

IMPORTANT:
DO NOT REUSE ANY ITEMS.
KEEP OUT OF THE REACH OF CHILDREN

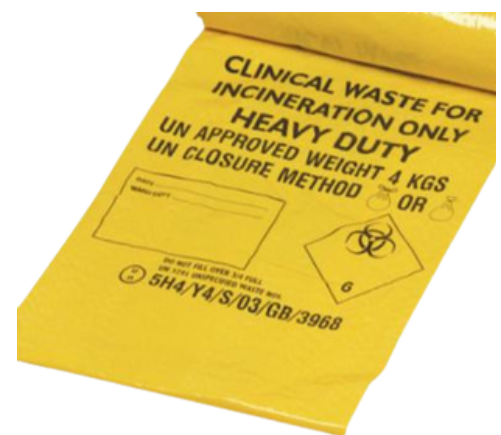

Three Step Spill Control

Absorb	Disinfect	Remove
		
<ul style="list-style-type: none"> • Put on gloves, apron and mask. • Apply Andarta Absorbent Powder uniformly over spillage until completely covered. • When liquid has congealed, dispose of using scoop and scraper and place in clinical waste bag. 	<ul style="list-style-type: none"> • Use absorbent pad and wipes with Andarta Cleaner and Sanitiser spray to cleanse the affected area. 	<ul style="list-style-type: none"> • Place the used scoop, scraper, gloves and apron into yellow waste bag and tie securely. • Dispose of the contaminated materials in an approved way. <p>IMPORTANT - DO NOT RE-USE ANY ITEMS.</p>